


ELECTRONIC ACUPUNCTURE

Handy as a ballpoint pen, you carry AcuZap with you everywhere; in your purse, in your desk, it's there when you need it. No batteries required.

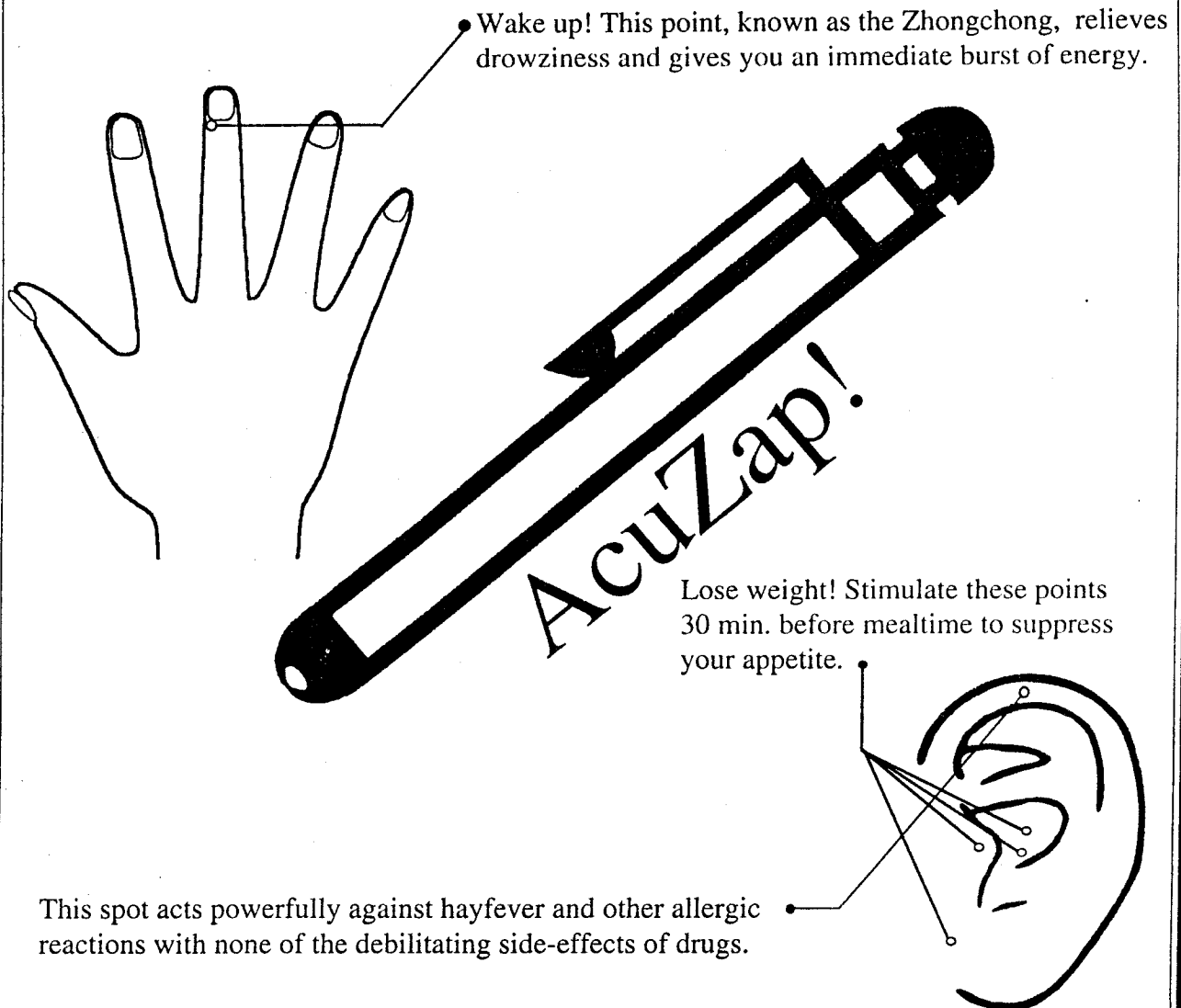

When a 2000-year-old healing art meets 20th century technology:

- **A BEAUTY AID:** By stimulating the nerves in your skin, AcuZap can help smooth wrinkles and improve your natural color; you will feel younger and look younger.
- **A RELAXANT:** The Acupuncture points, or "Tsubo" lie on "meridians" of energy that flow through your body. This energy can be either calmed or excited, depending on the points you choose to stimulate. Perfect for relieving stress after a long hard day.
- **A STIMULANT:** For sharpening the mind before that important meeting or exam. You can even use it through your clothes.
- **A GENERAL HEALTH TONIC:** The ancient Chinese system covers the entire spectrum of human health-related matters. It has proven effective against many problems that modern medicine cannot deal with, and can be used as a preventative regime to keep you in good health always.

➡ **NO NEEDLES!**

This piezoelectric pen stimulator is a painless, nonintrusive way to obtain all the benefits of acupuncture, even while wearing clothes. AcuZap generates a pulse of low-level electrical power with every push of the button; 0.000006 Amp. to be exact. Just place the tip of AcuZap over the "Tsubo" and press the top. Give yourself 15 or 16 jolts to thoroughly stimulate the point and get the energy flowing freely. For chronic problems, regular treatments work best.

➡ **NO BATTERIES!**

The piezoelectric charge is generated by the friction of the push-button and will work repeatedly and continuously, practically forever. The light weight and small size made possible by this design means that you can have your AcuZap handy at all times.

➡ **NO PROBLEMS!**

With AcuZap, you treat yourself. There is no need to visit a professional to make the ancient wisdom of Chinese medical practise work for you. And there is no need to pay the price. AcuZap is inexpensive, safe and effective. There are hundreds of points all over the body that can be used in the treatment of nearly any problem; learn them and you are on your way to a healthier, happier life. Use the product only as directed. Do not operate AcuZap in a wet environment as this may cause a malfunction.

Here are just a few of the disorders for which acupuncture has proven to be a specific and highly effective treatment:

- Stiff shoulders · Whiplash
- Nervous tic
- Toothache
- Hypertension
- Stomach · Duodenal ulcer
- Indigestion · Nausea
- Hepatitis
- Sprain
- Headache
- Neuralgia
- Sciatica
- Bronchitis
- Menstrual irregularity · Pain
- Eczema
- Insomnia
- Obesity
- Heart palpitation
- Backache
- Skin blemishes
- Arteriosclerosis
- Constipation
- Piles
- Gastritis
- Asthma
- Diarrhea
- Bursitis

Licensed by the Minister of Public Welfare in Japan. Medical Instrument Manufacture - Licence No. (01B) 0125
Registered Design No. 751106, 767643

For dealer information contact:


RICHWORLD COMPANY LTD.

1-2-12 TSUGIYA AMAGASAKI 661 JAPAN
PHONE:(06)497-3770 TELEFAX:(06)497-3779

AcheAway®

Electronic Acupuncture Pen
(Instruction Manual) PAT. #49720


Push Button

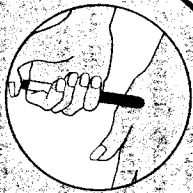


Thank you for your purchase of the **AcheAway®** Electronic Acupuncture Pen. For the most benefits from this unit, please read this manual carefully to familiarize yourself with its proper application.

Application Tip


Headache

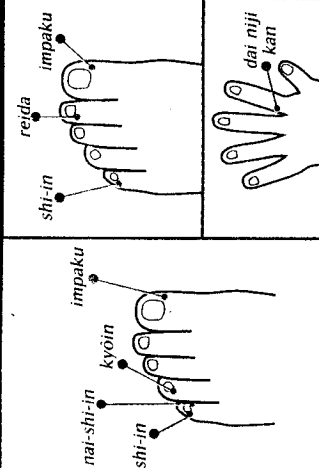

Constipation

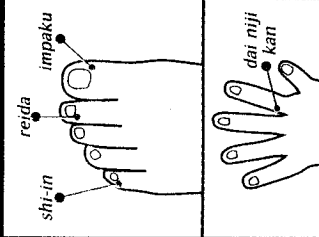


Sexual Strength/Frigidity

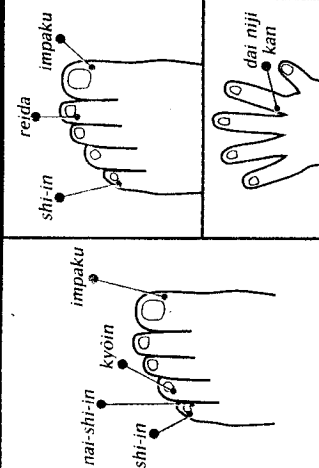

Hay Fever




Diet

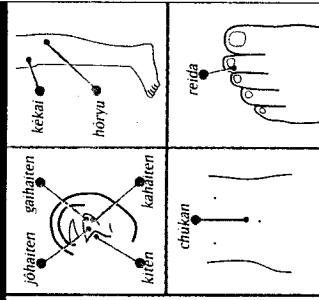


Hangover




Lumbago

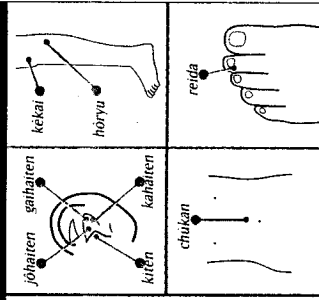


Anemia

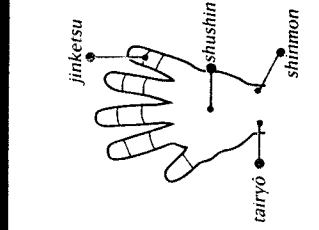


Eyestrain

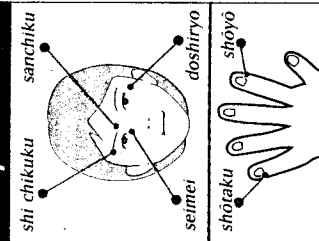

Fatigue

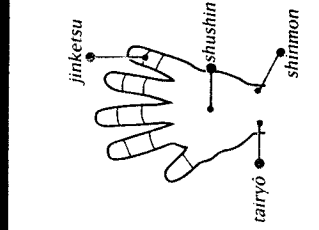

Toothache

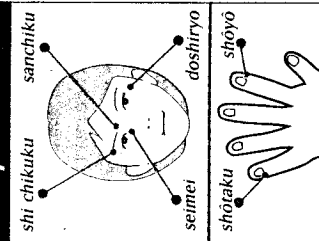

Acupuncture points are the spots along the pathways or meridians that run from the top of the head to the toes of the feet similar to the arteries and veins. However, acupuncture treatment of the back and the hips cannot be done by oneself. And if you are not an expert, it is very difficult to locate the acupuncture points on the body. The human body has a total of 12 energy pathways or meridians. About half of these points are concentrated at the fingers of the hands. The hands act like mirrors that are capable of showing various symptoms of diseases. Acupuncture points related to many diseases are concentrated at certain areas. Some parts of the body where essential acupuncture points exist are suitable for acupuncture self-treatment by any individual without requiring special skills.

While stimulating the acupuncture points with even a toothpick shows some effects, the Electronic AcheAway® Pen is a handy device specially designed to electrically stimulate the acupuncture points at the hands, feet, and ears. As most acupuncture points are located symmetrically on the left and right side of the human body, the effects of acupuncture treatment can be doubled by stimulating the points on both sides.

"Application of 10 electric pulse (clicks) with AcheAway® Electronic Pen is equivalent to 10 minutes of acupuncture or massage." Dr. Hashimoto, scientist


USEFUL INFORMATION

Although the concept of electrical stimulation may sound somewhat strange, you will soon appreciate the comfortable feeling it provides. Furthermore, you will welcome its positive effects.


NOTES

1. Do not use this device with gloves.
2. Avoid activating the device in the air, otherwise it could be damaged.
3. This device is good for approx. 100 applications a day. (The unit is warranted for 30,000 applications.) Typical application is 15-20 clicks per acupuncture point.
4. Low temperatures may degrade the performance of the device, however this is not a malfunction. Let it warm up to body temperature (e.g., in your pocket) before use.
5. Continued stimulation of the acupuncture points will lead to the desired results. (The effectiveness may vary between individuals.)
6. Avoid using this device during pregnancy.

Acupuncture points against shoulder strain

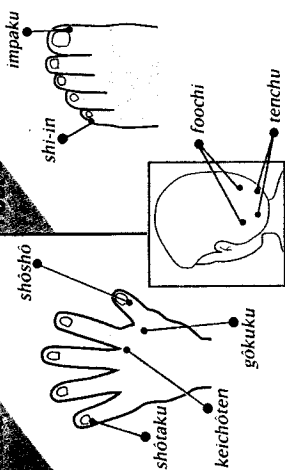

Acupuncture points against sleepiness

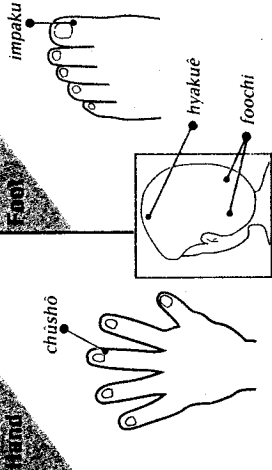

Inside the middle finger of both hands Inside the big toe of both feet

Since the Electronic AcheAway® Pen works with piezoelectricity, it creates an electric pulse when you push the button, without the need for batteries. Due to its extremely low current of 0.00006mA, it is inherently safe, while its pencil-like shape allows you to take it wherever you go.

As the acupuncture points are only approx. 0.2mm in size, they are very difficult to locate unless you are an experienced acupuncturist. However, the electric pulse generating section of the Electronic AcheAway® Pen is large enough that it can be easily applied to the acupuncture points, while simply pressing the pushbutton. About 15-20 applications is sufficient.

Made in Japan
Distributed By: